



pds *Healthcare*

Wheelchair User Guide
Flexicare and Easicare

(Refer to images on page 3)

No.	Product Features	Easicare	Flexicare
1	Attendant Brake Lock	•	•
2	User Brake	•	•
3	Seatbelt	•	•
4	Pusing Handle	•	•
5	Foldable Armrest	•	•
6	Side Buckle	•	•
7	Double-Cross Bar	•	•
8	Foldable Backrest	•	•
9	Leg Strap	•	•
10	Front Caster	•	•
11	Tyres (Two wheels option, flip to back cover)	•	•
12	Handrim	•	•
13	Back Pocket	•	•
14	Footplate	•	•
15	Tipping Bar (only for Easicare)	•	
16	Detachable Footrest		•
17	Double Layered Seat Cushion		•
18	Anti-Tipper		•

Features exclusive to Flexicare



Anti-Tipper



Swing-away /
Detachable Footrest



Double Layered
Seat Cushion

Parts of Wheelchair



Note: wheelchair shown above is Flexicare.

Usage of Attendant Brake

Caregiver may use the attendant brake to lock the wheels and stop the wheelchair.



It is advised to slow down the wheelchair before using the attendant brake.

To Activate



a) Grip the handlebar and press the brake handle up.



b) The brake is fully engaged when you hear a click

To Deactivate




a) Grip the handlebar and lift the brake lever to release the brake.



b) Use one of your fingers to release the brake lever slowly.

Usage of User Brake

The user brake helps the passenger to lock the wheel fully and stop the wheelchair from moving forward to prevent accidents from happening.

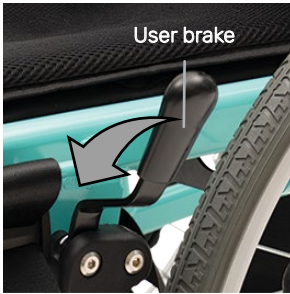

a) Push the user brake near the side panel forward.



The usage of the user brake can only be done when the wheelchair is not moving.



b) After activating the user brake, check if the back wheel is still able to move. If it cannot move, the user brake is fully functional.

Opening the Wheelchair



a) Lift up the handlebar until you hear a click sound.



To ensure your safety, please do not place your hand on the metal bar when you are folding up the handlebar.



b) After both the handles have been folded up, pull the handlebar to the side to open up the backrest.



c) Use both palms to push down the seat. Ensure the seat is flat before getting seated.



Note: keep your fingers pointing inwards and do not place them near the metal rod when pushing down the seats.



d) Swing the footplates down.

Closing the Wheelchair



a) Slide up both the left and right footplates.



b) Use both hands to grab the front and back of the seat and lift the center of the set up.




c) Finally, use the handlebars to push in towards the center to close the wheelchair completely.



d) If necessary, pull down the handlebar (Refer to foldable backrest instructions).

Usage and Operation of Wheelchair

Foldable Backrest

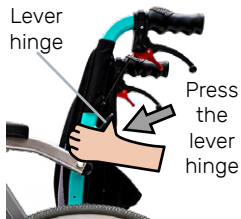

a) While holding the handlebar of the wheelchair, use your thumb to press down the lever hinge near the attendant brake.



For two-handed usage, hold on the handlebar with one hand and push the button down with the other.



b) Push down the handlebar while pressing on to the lever hinge. This will allow the backrest to be folded for ease of storage.

Flipping Up Foldable Armrest

Flipping up foldable armrests allows the user to be transferred out of the wheelchair in a convenient way.



Before flipping up the foldable armrest, check and ensure there are no hands near the foldable armrest. If there is anyone sitting on the wheelchair, please ensure that their body is not near the foldable armrest.



a) Unclip the side buckle that is connected from the seats to the foldable armrest.



b) Grab on to the front part of the foldable armrest and push it up.




c) Swing the foldable armrest to the maximum height (90 degrees vertically) and ensure it does not fall after letting go.


Swing Footrest Inwards/ Outwards

(Feature exclusive to Flexicare)

a) Pull up the footrest lifter, and push the footrest inwards/ outwards.

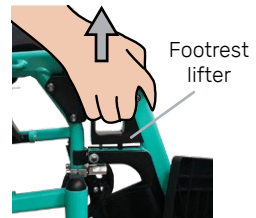

b) Swing the footrest back to original position.

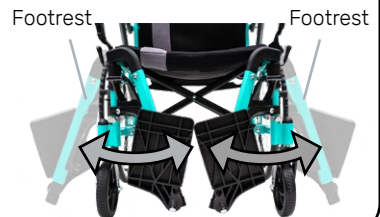

Detachable Footrest

(Feature exclusive to Flexicare)

a) To remove, grab the footrest lifter and turn the footrest outwards. Pull up the footrest lifter and remove the footrest.

b) To place it back, align the footrest with the wheelchair pole and place it vertically downwards. Pull the footrest lifter and twist it inwards.



Adjustable Anti-Tipper

(Feature exclusive to Flexicare)

The anti-tipper is used to prevent the wheelchair from tipping backwards for the passenger's safety.



a) To remove the anti-tipper, pull up the metal rod vertically and remove the anti-tipper.

b) To put it back, insert the anti-tipper horizontally to the wheelchair and align both the holes. Put the metal rod inside the holes and ensure the anti-tipper is secured.





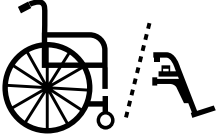
Double Layered Seat Cushion

(Feature exclusive to Flexicare)



a) To remove the canvas, please find the velcro of the seats and tear it off. To put it back, simply reverse the steps.

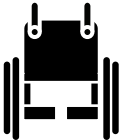What's Inside the wheelchair kit

Assembled Wheelchair	Simple Tools	Wheelchair Manual	Warranty Card	Removable Footrests (Only for certain models)
				

Wheelchair Safety

To prevent unnecessary injuries caused by using an unfamiliar product, please practice wheelchair movements in a spacious environment.

For users:




- Ensure the screws and bolts have been secured tightly before using the wheelchair.
- Make sure the brakes are locked before mounting or dismounting the wheelchair.
- Ensure that its in an operational condition before using the wheelchair.
- Keep the seat cushion and hand rests away from prolong exposure to sunlight.
- Do not drink and operate the wheelchair under influence of alcohol.
- Be constantly aware of your body movement and center of gravity to avoid falling.
- When going down slopes, avoid facing forward to prevent accidental falls.
- The wheelchair can support one person, with a maximum load of 110kg.
- Do not step on the footplate to dismount the wheelchair.

For attendants:

- Before moving, ensure that the user and attendant brakes are not active.
- Ensure the patient's feet are resting on the footrest, and that their elbows are tucked in and resting on the armrest.
- Avoid steep or uneven surfaces to prevent the wheelchair from tipping over.
- Always inform the person in the wheelchair your next planned move, such as tilting them or changing direction.
- Do not hang heavy items on the handle bars to prevent wheelchair from tipping.
- Before moving off, ensure the passenger is wearing the seatbelts.

PDS Wheelchair Specification

Specification	Easicare	Flexicare
Frame type	Aluminum	Aluminum
Armrest pad	PU	PU
Back wheel	22"PU tyres (Wheelchair)	22"PU tyres (Wheelchair)
	16"PU tyres (Pushchair)	16"PU tyres (Pushchair)
Front caster	6" PVC	6" PVC
Seat width	46cm/ 18" inch	46cm/ 18" inch
Seat depth	40cm/ 15.7"inch	40cm/ 15.7" inch
Armrest height	47cm	47cm
Full Width (unfolded)	65cm	65cm
Width (folded)	30cm	30cm
Seat height	40cm	40cm
Back rest height	110cm	110cm
Max user weight	110kg	110kg
Wheelchair Weight	12kg (Pushchair)	14kg (Pushchair)
	13kg (Wheelchair)	15kg (Wheelchair)

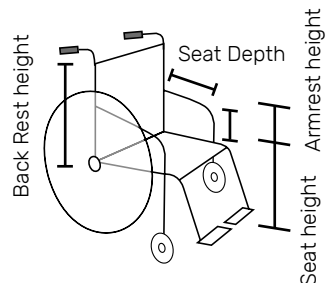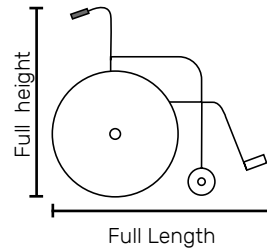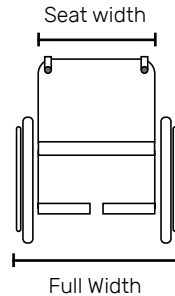

There are two options of the back wheel, you may only choose 1 at the point of purchase:

Option 1 - Wheelchair rim 22" inch

Option 2- Pushchair rim 16" inch

Company Information



Tel: (65) 6776 6200

Fax: (65) 6776 6882

E-mail: pds@safety.com.sg

Address: 10 Pandan Cres, #05-03/04 (LL2), Singapore 128466

Operating hours: 8.30am to 6pm/ Monday to Friday