


# Druglizer

## Druglizer Cartridge & Oral Fluid Testing



It is recommended the subject **not consume food, beverages or smoke at least 10 minutes prior to oral fluid (saliva) collection.**

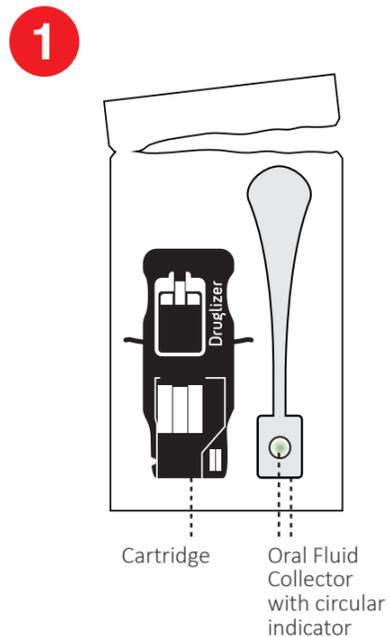

To encourage saliva production, instruct the subject to press the inside of their cheeks with the tip of their tongue four times.

Tear open the pouch containing a Druglizer Cartridge and Druglizer Oral Fluid Collector.





Turn the LE5 Drug Tester on. The "Insert Cartridge" message will display. Take the Druglizer Cartridge and insert it into the top of the LE5 Drug Reader Module.

The flat area should face back while the area with the collector recess should face towards the front. When inserted the screen will display "Insert Collector".



Instruct the subject to insert the Collector into their mouth and rub the pad onto the inside of both cheeks as well as under the tongue and around the mouth.

**IMPORTANT: LE5 Drug Tester training MUST be undertaken in order to ascertain correct saliva collection. Collect saliva until the Collector indicator changes to a BLUE that is consistent from edge to edge:**

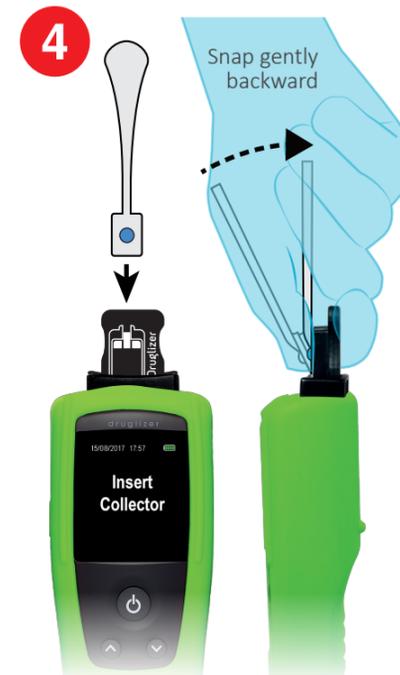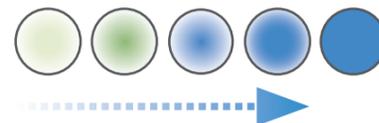

Position the Collector vertically with the pad facing downward. **With your thumb on the indicator**, insert into the Druglizer Cartridge recess and snap (gently backward) into place.

The LE5 Drug Tester will automatically detect the wetted Collector and begin analysing.


**Position the LE5 Drug Tester horizontally – preferably on a flat surface. Orientation alerts will help you from moving or tilting the device whilst it is analysing a test sample.**



Results will be displayed on screen from 90 seconds – they are updated as they are calculated and may display:

- Negative (green)**
- Unconfirmed (red)**
- Invalid (grey)**

Results can be accessed from the LE5 Drug Tester at a later time.

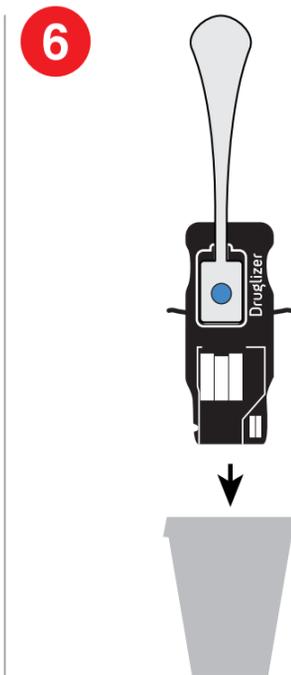

After each test, pull out both the cartridge and collector and dispose of with other waste as per local waste disposal requirements.

The LE5 Drug Tester will automatically reset and be ready to insert a new Druglizer Cartridge by displaying "Insert Cartridge".